

TubusOne mouthpiece should be replaced every day for optimal hygiene




Clean TubusOne daily



TubusOne tip should be replaced every 4-6 week and mounted correctly – all the way down over the thin stick



Do not use dishwasher or any chloride based cleaning products

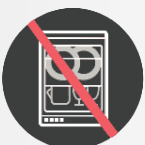


Remember

TUBUS one

Cleaning guide



Aluminum tube 1
Piston 2
Tip 3
Cleaning rod 4
Mouthpiece 5
TubusClip 6

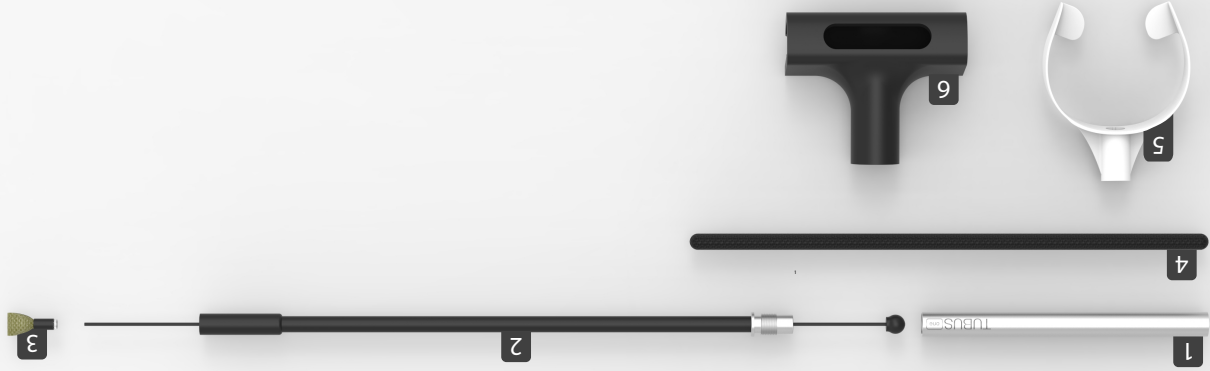

▶ See [videoguide at tubusone.com/get-started](https://tubusone.com/get-started)

Cleaning TubusOne

Assembly of parts

Check before use

- Check that TubusOne is dry before assembly
- Insert the thin stick into the piston (2) from the end with thread
- Tip (3) must be attached to the thin stick. Remember to push the tip (3) all the way down. Press to ensure it is mounted correctly

- Turn the aluminum tube (1) from the piston (2) counterclockwise
- Insert cleaning rod (4) into aluminum tube (1) and move back and forth. Wash aluminum tube (1) in water and soap. Remember to rinse in clean water
- Remove tip (3) and pull the thin black stick out of the piston (2). You now have 3 loose parts including a tip (3). Remember to rinse the piston (2) in water
- Wash the round part in water and soap. Rinse in clean water – use alcohol if necessary

- Check that the round end does not exceed the hole in the mouthpiece when the tube is held vertically in front of you. If it is visible, the tip has to be pressed further down on the stick
- Check that the mouthpiece is mounted correctly on the aluminum tube with both holes facing downwards
- Check that the tip has been pressed correctly down on the thin stick
- If TubusOne does not work, try to replace the tip and/or the mouthpiece or clean the product and try again